

Product Instructions From American Medical Industries™

Instructions on using the EZ-Swallow Pill Crusher




1. Grasp the Pill Crusher portion (bottom) in one hand. While holding the bottom, turn the storage container (upper portion) counter-clockwise until the two units are separated from each other.

2. Insert the pill(s) to be crushed in the bottom of the unit so they are positioned on the grinding surface.



3. Replace the top portion and begin screwing the top clockwise into the bottom until the top and bottom portions of the unit come into contact with the pills. At this point a slight increase in pressure will be needed to continue turning the units together as the pill is crushed between the grinding surfaces. For best results, turn the top unit clockwise a bit to release the pressure on the pill pieces. Shake the unit to reposition the pieces being ground. This will help prevent medications with high moisture content from being ground into a paste. It will also make grinding very hard medications like calcium pills into a fine powder much easier.

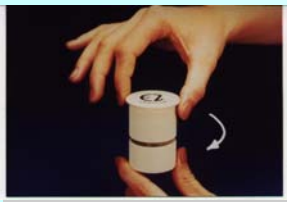

4. Separate the two units. Pills will be ground into a fine powder and may be mixed into soft food or liquids.

5. If desired the lid may be removed to expose the large storage area. Extra pills can be placed there when traveling or for convenience. This area can also be used to mix crushed medications with liquids or soft food.



6. It is recommended that the unit be cleaned thoroughly between uses to prevent cross contamination of medications. Use warm soapy water. This unit is also dishwasher safe.