

Product Instructions From American Medical Industries™

Instructions on using the EZ-Swallow Pill Splitter



1. Open lid on the pill splitter and insert the pill in the diamond ring. Position the pill so that it touches both sides of the ring. An easy way to do this is to tip the splitter and allow the pill to slide to the top or bottom of the diamond ring.
2. Lower the top and snap it closed.
3. Open the top of the pill splitter to remove the cut pill sections.
4. Store unused portions in the handy storage base.
6. It is recommended that the unit be cleaned thoroughly between uses to prevent cross contamination of medications. Use warm soapy water. This unit is also dishwasher safe.