

EZ-Irrigator Wound Irrigation System

Description

WI-30 Consists of 30 cc Irrigation Syringe with pre-attached two-way check valve and 72" spiked infusion line. WI-10T contains a 10cc self-filling syringe for single handed irrigation technique.

Indications for Use:

The EZ-Irrigation Systems are intended for the irrigation of wounds and must be used in conjunction with the tunnel irrigation tip and SoftShield Irrigation tips.

Contraindications:

This device is not intended for uses other than those indicated.

Warnings and Precautions

Single patient reuse of this device is contraindicated when sterile technique is indicated.

Use of the device on multiple patients may result in cross contamination and the spread of infection.

Federal Law (USA) restricts this device to sale by or on the order of a licensed practitioner.

Do not use this device if package or product shows signs of damage.

This device is for single patient use only.

This device is to be discarded, following institution protocols, after use.

When using the SoftShield, the following precautions must be followed:

- During the irrigation, always position shield so that its rim rests lightly or as near as possible to the irrigating surface.
- Do NOT press the shield onto the skin to create a watertight seal. Allow the fluid to run out from the bottom of the shield.

Directions for use:

- Remove the selected Irrigation system from sterile package. Make sterile connection of spike into irrigating solution bag.
- Connect the SoftShield (See precaution for SoftShield above.) to the male luer of the two-way valve. Prime the device by repeatedly pumping the syringe plunger.
- To irrigate, position shield as required, and depress syringe plunger.

Preparation of the device for storage after use.

- Gripping only the female luer connector of the SoftShielded, remove shield. NOTE: During removal do not pull on shield dome.
- To flush the system, withdraw the plunger from the syringe and fill the syringe with irrigating fluid.
- Depress the syringe plunger, dispensing fluid into a proper receptacle.
- Wipe the male luer tip of the 2-way valve with 70% isopropyl alcohol (not supplied).
- Remove sterile cap from its packaging and fit snugly over the male luer of the 2-way valve.
- Store in a cool, dry place until the next irrigation procedure. The wound irrigation system may be re-used on the same patient for 14 days, as long as a new SoftShield is used for each irrigation procedure and the device is properly stored and handled per instructions.

Preparation of the Device for re-use

- Remove and discard the protective cap on the male luer of the 2-way valve.
- Withdraw the plunger in the syringe, completely filling syringe with irrigating solution.
- Press the plunger into the barrel, dispensing the irrigating fluid into a proper receptacle.
- Attach the female luer of the SoftShield to the male luer of the 2-way valve.
- Begin irrigation following directions for use above and institution protocol.



American Medical Industries, LLC

336 E 4th Street

Dell Rapids, SD 57022

Ph: 605-428-5501 * Fx: 605-428-5502

info@pillcrusherguys.com

www.pillcrusherguys.com

Syringe Model	Irrigation Pressure	Depress Syringe Time
30cc	8 psi	8 sec
10cc	8 psi	3 sec